The **KMINA WHEEL CHAIR RAMP** is used to overcome obstacles up to 1/4" in height, especially for transition profiles between two types of floors, although it can also be used to overcome cables or other objects.

INSTRUCTIONS FOR USE

The initial size of the ramp is 35.5" and the cutting slots are every 2", which will allow the ramp to be cut to a minimum of 23.5". 1. Measure the gap in your door frame and select the groove that best fits the door frame. 2. Using a cutter, knife or blade, use the guide to cut to the desired size. 3. Position the ramp between the door frame.

WARNINGS

- Any serious incident involving the product must be reported to the manufacturer and to the competent authority of the Member State in which the user and the patient are established.
- Do not remove the sticker provided on the product. If necessary, note the batch of the product so that the manufacturer can be notified in the event of an accident.
- Inspect periodically for any wear or damage.

CLEANING AND MAINTENANCE

Clean with a damp cloth, avoid alcohols or solvents and avoid exposure to direct sunlight for long periods of time.

WARRANTY

KMINA warrants this device to be free from defects. The warranty will remain in force for 3 years from the date of original purchase by the consumer. The warranty does not extend to parts exposed to natural wear and tear that require periodic replacement.

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