KMINA - PRO SELF-PROPELLED COMMODE TOILET CHAIR USER MANUAL

KMINA - SILLA PRO AUTOPROPULSIÓN - MANUAL DE USUARIO
KMINA - PRO SELBSTAHRENDER TOILETTENSTUHL - BENUTZERHANDBUCH
KMINA - SEDIA PRO SEMOVENTE - MANUEL D'UTILISATION
KMINA - CHAISE PRO AUTOPROPULSÉ - MANUEL D'UTILISATION

KMINA
#WalkTheFuture

The **KMINA PRO SELF-PROPELLED CHAIR** is designed to facilitate the daily hygiene of people with mobility problems and to make this task as easy as possible for both the user and the caregiver. The chair can be used in combination with the toilet and in the shower, and can be steered by a patient with sufficient autonomy thanks to its self-propelled wheels, or pushed by the caregiver. Keep these instructions for future reference. This chair is in compliance with EU Regulation 2017/745 concerning medical devices.


ASSEMBLY

1. Actuate the locking brake as shown in Figure 1 by pressing the lever downwards. This prevents the frame from moving during assembly. To release the brake, push the lever upwards.

2. Insert the iron into the central hole of the wheel and place it on the frame by pressing the iron button in order to fit the wheel completely, as in Figure 2. Make sure that the driving rim is facing outwards.

3. Place the brakes of the self-propelled wheels by inserting them into the central structure at a distance of 4 cm from the top of the tube. Use the Allen wrench provided to tighten the screws. Tighten them alternately so that they are completely fixed and pull the lever to activate the brake.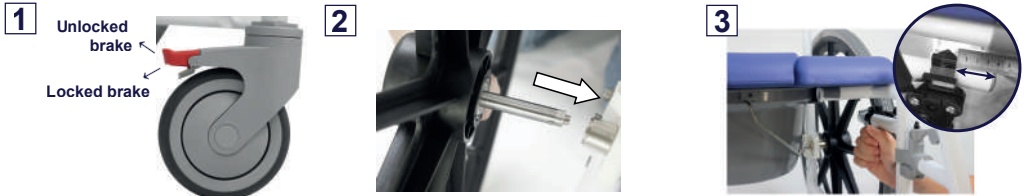


4. Unscrew both locking screws on the frame and insert the backrest tubes into the frame as shown in Figure 4. Retighten the locking screws and then move the armrests downwards into position for use.

5. Take the seat cushion and push it with the recess in the rear cross bar of the frame and once positioned press the seat firmly down into the frame until it is fixed. Make sure that the clips on the underside of the seat fit fully into the frame.

6. To attach the footrests, insert them into the tube (1) and rotate them forward (2) as shown in Figure 6. You can use the footrest locking lever to lock and unlock the footrest so that the footrests can be left open to exit. They can also be easily removed and replaced.

7. To adjust the height of the footrest, unscrew the connection at the back of the footrest tube (1) as shown in Figure 6, set the correct height (2) and retighten the screws (3). **IMPORTANT:** The adjustment is correct if the user's legs form 45° when placing the feet. You can lift the footrest base if you wish to get out of the chair and easily fold down again.

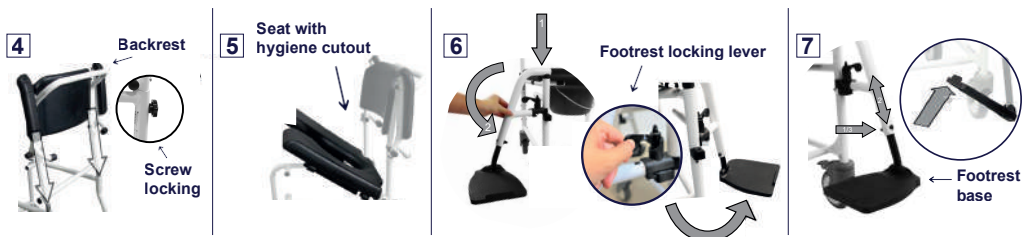1. USE OF THE BUCKET CHAIR

Before use:

- engage the locking brakes and remove the seat opening cover.
- Place the bucket without the cover, under the seat.
- Push the bucket from behind until it locks in place.

After use:

- Pull back to empty.
- Replace the lid.
- After emptying, clean as described in (Cleaning and Care).

2. USE OF THE CHAIR WITH CONVENTIONAL TOILET

Before use, check that the height of your toilet is lower than the maximum height allowed (43 cm).

- Pull out the bucket from the rear.
- Remove the seat opening cover.
- Lift the toilet lid.
- Place the chair in the toilet opening and engage the locking brakes.

3. USING THE CHAIR IN THE SHOWER

- Engage the locking brakes.
- Pull back and remove the bucket.
- Remove the seat opening cover for cleanliness and hygiene. **IMPORTANT:** Do not attach the bucket while the user is still seated in the shower chair.

TECHNICAL SPECIFICATIONS

LENGTH	WIDTH	HEIGHT
40"	25"	37"

USER MAX. WEIGHT	WEIGHT	Ø DRIVE WHEELS (x2)
265 lb	37.5 lb	24"
SEAT HEIGHT	SEAT MEASUREMENTS	LENGTH WITHOUT LEGREST
21"	17.3"x16.5"	27.5"

MATERIALS

Frame: Aluminum; Seat, backrest, armrests, Tire: PU; Footrest, Magnetic wheel: Nylon + fiberglass; Bucket: PP.

Pre-use checks

- Please check that all parts are connected, have been correctly tightened and that the brakes work.
- Check that the seat is securely fastened.
- You must not use the KMINA PRO wheelchair to transport several persons or loads. Observe the appropriate uses.
- Make sure that all parts are secure and that moving parts function properly. IF you find signs of wear or damage to your chair, **DO NOT USE IT.**
- Do not exceed the maximum user weight: 265 lb.

During use

- All wheels must be in contact with the ground at all times and on a flat, smooth surface. Use ramps to cross any obstacles.
- The KMINA PRO chair is designed for use indoors, bathrooms and showers.
- Be careful if arms are flapping as clothing or body parts may get caught in the armrests.
- Do not stand up on the footrests to get in or out of the chair, or in a stationary position.
- Do not allow infants, small children and persons who do not understand the operation to use this product by themselves.
- The brakes must be in the locked position before the uses explained above, including in the stationary position. Please note that the risk of tipping may increase in the case of weight shift when moving your body. If the locking brake is damaged or not usable, the wheelchair must not be used.
- Any serious incident involving the product must be reported to the manufacturer and to the competent authority of the Member State in which the user and patient are established.

CLEANING AND CARE

- Clean and disinfect the shower chair and toilet or bucket after each use with running water and conventional disinfectants.
- The frame parts and upholstery can be cleaned with a damp cloth. In the case of the wheels as well, including a plastic brush (do not use wire brushes!).

WARRANTY INFORMATION

- KMINA warrants this chair to be free from defects. The warranty will remain in effect for 3 years from the date of original consumer purchase. The warranty does not extend to parts exposed to natural wear and tear that require periodic replacement, such as brakes, wheels or upholstery. The warranty does not cover:
 - Replacement parts supplied by anyone other than KMINA or the authorized dealer.
 - Any failure of the unit during the warranty period that is not caused by a defect in material or workmanship, or if the failure is caused by misuse.
 - KMINA is not liable for accidental damage or property damage.
- In case of return, properly pack the chair preferably in its original packaging, otherwise KMINA will not be responsible for possible damage caused during shipping.

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