**USER MANUAL
MANUAL DE USUARIO
BENUTZERHANDBUCH
MANUEL D'UTILISATION
MANUALE UTENTE**

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**KMINA - ELECTRIC STAIRLIFT
KMINA - SILLA SALVAESCALERAS ELÉCTRICA
KMINA - ELEKTRISCHER TREPPENLIFT
KMINA - MONTE-ESCALIER ÉLECTRIQUE
KMINA - MONTASCALE ELETTRICO
KMINA - 電動階段昇降機**

	ENGLISH	ESPAÑOL	DEUTSCH	FRANÇAIS	ITALIANO	日本語
A.	Rear bars	Barras traseras	Hinteren Stangen	Barres arrière	Barre posteriori	リアバー
B.	Structure	Estructura	Struktur	Structure	Struttura	構造
C.	Electric track	Oruga eléctrica	Elektrische kette	Piste électrique	Traccia elettrica	電気トラック
D.	Motor	Motor	Motor	Moteur	Motore	モーター
E.	Wheels with brake	Ruedas con freno	Gebremste Räder	Roues freinées	Ruote con freni	ブレーキ付き後輪
F.	Wheels	Ruedas	Räder	Roues	Ruote	ホイール
G.	Footrest	Reposapiés	Fußstütze	Repose-pieds	Pedana	フットレスト
H.	Front bars	Barras delanteras	Vorderen Stangen	Barres avant	Barre anteriori	フロントバー
I.	Lithium battery	Batería de Litio	Lithium-Batterie	Batterie au lithium	Batteria al litio	リチウム電池
J.	Seat belt	Cinturón	Sitzgurt	Ceinture de sécurité	Cintura di sedere	シートベルト
K.	Arms	Brazos	Arme	Bras	Bracci	武器
L.	Lift bar	Barra elevadora	Hebebügel	Barre de levage	Barra di sollevamento	リフティングバー
M.	Motor control	Mando de control	Motorsteuerung	Contrôle du moteur	Controllo del motore	モーター制御
N.	Headrest	Reposacabezas	Kopfstütze	Appui-tête	Poggiatesta	ヘッドレスト
O.	Loader	Cargador	Ladung	Chargeur	Caricatore	ローダー
P.	Keys	Llaves	Tasten	Clés	Chiavi	キー

EN | To turn on the electric stairlift, you must press the power button, it will illuminate. To check the battery level, look at the display on the motor control.

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EN | To charge the battery, locate the battery keys (P) located on one of the bars of the frame. Insert one of them into the slot and turn clockwise until the battery is removed.

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EN | Unscrew the battery connection cable anticlockwise. Plug in the charging cable and charge the battery.

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EN | When the battery is charged, remove the charging cable and screw it clockwise by hand as it was at the beginning.

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EN | Pull the seat of the electric stairlift towards the floor until you hear a click when the seat locks. The stairlift has two positions.

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EN | Pull the rear bars (A) firmly upwards and the footrest (G) and armrests (K) firmly downwards into position.

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EN | Press up on the parallel bar and lift the lift bar (L) until it locks into position. 3 heights can be selected, depending on the height of the user.

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EN | Unlock the black tab on the side of the rear centre bar to the side.

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EN | Press the red lever to release the electric tracks backwards until you hear a click, checking that they are in the locked position. The electric tracks have two positions.

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EN | If it is necessary to support the user's head during transport, the headrest (N) can be used to support the head.

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EN | Press down on the red bottom locking bar by hand. While the bar is depressed, push it into the frame. The electric stairlift will begin to fold.

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EN | Re-fold the tracks in reverse order. Return the rear bars (A), footrest (G) and arms (K) to their original position.

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IT | Luce.

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UPSTAIRS /HACIA ARRIBA/ OBERGESCHOSS/ VERS LE HAUT/
VERSO L'ALTO/ 上へ

EN | With the user fastened with the seat belt, place the stairlift on the edge of the stairs and press the power button on the motor control. Keep the wheelchair at the angle of the electric tracks, always with two points of support, by pressing the up button. The stairlift will move upwards. If you need to stop halfway, release the up button. When you reach the last step, release the button and slowly raise the stairlift.

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EN | With the user fastened with the seat belt, place the chair on the stairs with the front wheels on the edge and press the power button on the hand control. Then press the lowering button. Lean slightly towards the angle of the electric tracks and the stairs, always with two points of support. The caregiver must use downward force so as not to lift the chair. When the chair reaches the last step, release the button and lift the wheelchair slowly.

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The **KMINA electric stairlift** has been designed as an aid for transporting people up and down stairs in buildings. It has a sliding system with electric tracks for safer and easier transfer. It can be used in hospitals, emergency centres, fire brigades, emergency ambulances or as a transport aid for family use.

FEATURES

- Materials: High strength aluminium alloy.
- Wheels: 5 7/4". Rear wheels with brakes.
- Lift bar with 3 positions to assist the operator in moving the stairlift up and down the stairs.
- The user can adjust the tension of the sliding system.
- Maximum user weight: 350 lb.

WARNINGS

- Any serious incident involving the product must be reported to the manufacturer and to the competent authority of the Member State in which the user and patient are established.
- Never leave an evacuee unattended in the electric stairlift and always use the safety system.
- Avoid changing the evacuee's position quickly, as this could result in injury. Hold the electric stairlift while adjusting the handles.
- Improper maintenance may cause injury. Keep the electric stairlift only for the purposes described in this manual.
- Read the instructions before installation.
- Confirm that all components and parts are properly assembled and screwed together before use.
- In the event that the product runs out of battery power during a manoeuvre, remove the front bars (H) for manual operation.
- It is essential that the operator is trained by a qualified instructor before using the stairlift chair.
- The stairlift chair may involve a higher degree of risk and require greater skill from the operator.
- The use of the stairlift chair is dangerous if the user does not know or follow the manufacturer's instructions for use.
- Product stability tests have been performed using test dummies and standard test wheelchairs, stability performance may vary in real-life situations.
- This product should not be used on spiral staircases, curved arched staircases, steps with an angle of 70° or more, and trapezoidal surfaces with rough and uneven surfaces.
- The stairlift chair must never be stopped in contact with only two steps. Temporary stops must be made in such a way that the tracks are in contact with the front of at least three steps.
- The user in the stairlift chair must be securely fastened with the seat belt fastened.
- This product is limited to one person at a time, be sure to wear a seat belt when using.
- This product should not be used by children or pets.
- Do not exceed the maximum weight limit of 353 lb.
- Use the original battery, charge it in a dry environment.
- Do not open or replace the battery yourself and dispose of the battery according to the disposal laws of each country.
- Do not force the use of defective or badly worn stair climbers to avoid hazards.
- The operator must not release the armrest with both hands at any time.
- If the operator cannot fully control the device by himself, there must be an assistant in front of the vehicle to keep the chair balanced.
- When descending stairs, the front wheels of the power stairlift should be kept as far away from the edge of the steps as possible.
- This product is not waterproof and may only be used indoors. Keep away from humidity and fire.
- This product must not be disassembled or its accessories modified, as this may cause electric shock or damage to parts.

MOTOR AND BATTERY

- Motor output power: 120 W
- Motor output voltage: 24 V
- Battery voltage: 24 V
- Nominal capacity: 11.6 Ah
- Charging current: 2 A
- Charging time: 6 h

CLEANING AND MAINTENANCE

Clean the electric stairlift and its parts with a clean cloth or soft brush and warm soapy water. Rinse with lukewarm water and allow to air dry. Do not clean with corrosive products. If necessary, a stiff bristle brush may be used.

WARRANTY

KMINA guarantees that this device is free from defects. The warranty shall remain valid for 3 years (except motor, battery and motor control valid for 1 year) from the date of original purchase by the consumer. The warranty does not extend to parts exposed to natural wear and tear that require periodic replacement. This warranty does not cover:

- Replacement parts supplied by anyone other than KMINA or the authorised dealer.
- Any failure of the unit during the warranty period that is not caused by a defect in material or workmanship or if the failure is caused by misuse.
- KMINA is not liable for accidental damage or damage to property.

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