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**INSTRUCTIONS FOR USE, STORAGE AND WARRANTY**

Dear Customer,  
Thank you very much for placing your trust in an Orliman product. Please read the instructions carefully. Keep these instructions and the packaging for future reference. If you have any questions or concerns, please contact your doctor, orthopaedic specialist or our customer service department.

ORLIMAN S.L.U. guarantees all its products as long as the original configuration has not been manipulated or altered except for the intended use as described in these instructions.

If the products are used in combination with other products, replacement parts or systems, make sure they are compatible and made by Orliman®. It does not guarantee any products with altered characteristics due to improper use, defects or breakage of any kind. The statutory regulations of the country of purchase apply. Please first contact the retailer from whom you obtained the product directly in the event of a potential claim under the warranty. If any serious incidents related to the product occur, notify Orliman S.L.U. and the corresponding competent authority in your country.

Orliman would like to thank you for choosing this product and hopes you a speedy recovery.

**REGULATIONS**

**MD** This article is defined as a class I medical device. A Risk Analysis (UNE EN ISO 14971) has been carried out, minimising the existing risks. Tests have been in accordance with European Regulation UNE-EN ISO 22523 on Prostheses and Orthoses.

**INDICATIONS**

**HOT:** Contractures, Aches, Lumbagos, Arthritis (stiffness in the joints), Cramps...

**COLD:** Hematomas, Sprains, Strains, Headaches, Arthritis (inflammatory), Insect bites, Tooth pain...

**FITTING INSTRUCTIONS**

When fitting the product, you must adhere to the following instructions:

**HOT:**

1-Leave the compress in a cool dry room.

2-Heat the compress:

· MICRO-WAVE

Respect heating time indicated on the compress.

· BAIN-MARIE

Put the compress in boiling water for 5 min.

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**COLD:**

For the first use:

1-Put the compress 2h minimum in the freezer.

2-Remove compress from freezer.

3-Put the compress in its protection.

4-Place it on the painful area during 20 min.

5-Wait for 30 min before using it again.

**PRECAUTIONS**

**Carefully read the information on the compress!**  
For people sensitive to heat or cold, or suffering from poor blood circulation or neuropathy, please consult your doctor before use. Do not use the compress without its protection.

Overexposure to cold can cause a cold burn. Do not apply the cold compress directly to the skin. In case of unusual sensation or pain, stop the cryotherapy session immediately and consult a healthcare professional.

This product is contraindicated in cases:

- Allergy to cold,
- Raynaud's syndrome,
- Nerve degeneration
- Severe diabetes,
- Circulation disorder (deep thrombosis, occlusive arteriopathy, patient with weak peripheral circulation),
- Arterial hypertension.

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**EN** The manufacturing date is included in the batch number which can be found on the packaging label as **LOT**, in the following way: the second and third digits represent the year of manufacture and the fourth and fifth digits represent the month.

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