## EN

### PRESSURE REDISTRIBUTION PADS INSTRUCTIONS FOR USE

#### INTENDED PURPOSE

Fresco Pads is a range of pressure redistribution polymer gel pads available in various sizes, thicknesses and anatomical shapes which help to reduce the risk of pressure damage, when used as part of a pressure ulcer prevention strategy.

Fresco Pads conforms in shape to its physical environment, and any pressures applied to it, enabling patient immersion and envelopment and redistributing peak pressures over a wider surface area.

It can be used anywhere on the body which is identified as being 'at risk', especially where there are bony prominences. It is intended for single patient use and can be washed and re-applied to the same patient.

#### INDICATIONS

Fresco Pads is indicated to help prevent pressure damage occurring for patients identified as being 'at risk' and can be used for recently healed damage to protect against the risk of recurrence. Pads should only be used on intact skin and can be cut to size to ensure best fit. The range of shapes and sizes provides pressure redistribution for different anatomical areas: Fresco Pads, Strips and Tubing can be cut to size. Please see appendix 1 for the full range.

Pads	Strips	Sacrum	Heel	Tubing
Occiput Between the knees Under the chin Hand contractures	Digits Longer body areas Medical devices	Sacrum Ankles Shoulders	Heels Elbows	Medical device tubing and straps – Oxygen tubing, nasal cannula, face masks

#### CONTRA-INDICATIONS

Do not use on broken skin.  
Do not obstruct airways.  
Do not apply over wound dressings as this may compromise the breathability of the dressing and pressure redistribution properties of Fresco Pads.  
Do not apply to mucous membranes.  
For external use only.  
Single patient use.

#### WARNINGS AND PRECAUTIONS

Inspect the skin at regular intervals. Discontinue use and seek healthcare professional advice if:  
a) Loss of skin integrity occurs ie breakdown.  
b) Deep tissue injury is suspected or confirmed.  
c) Any skin irritation is present.  
d) Heat rash or reddening of the skin occurs.

Before application and reapplication, ensure the skin is completely dry.  
Fresco Pads contains a high grade medical mineral oil which may leach onto absorbent surfaces. Protect non-culporid fast fabrics.  
Fresco Pads contains an acetate mineral oil which may leach onto absorbent surfaces. Protect non-culporid fast fabrics.

Plastic clamshell packaging can be used to store Fresco Pads during hygiene processes or for safekeeping.

If cutting to size, ensure this is performed away from the patient to prevent accidental contamination from foreign bodies.

Fresco Tube can be used on medical tubing with varying diameters:  
- Secure in place with tape to avoid slippage.  
- Always position Fresco Pads tubing against the skin.  
- Multiple tubes can be used to ensure full coverage of the device.  
The edges of Fresco Pads should extend beyond the margins of the at-risk area.  
Do not use under compression garments i.e. multi-layer bandaging and compression hosiery. Replace when it appears visibly worn, becomes difficult to clean, or fails to provide complete coverage of the area at risk of pressure damage due to loss of integrity.  
Do not activate or place in direct sunlight.

#### PATIENT TARGET GROUP:

Any person at risk of pressure damage.  
Individual risk assessments should be made to assess the suitability of any medical device.  
Single patient use.

#### INTENDED USERS:

Intended for use in home and professional healthcare facilities by medically trained and untrained users.

#### INSTRUCTIONS FOR USE

- Select the most appropriate size and shape to ensure adequate protection.  
a. The edges of Fresco Pads should extend beyond the margins of the at-risk area by at least 10mm.
- Clean and thoroughly dry the skin.
- Remove from the packaging and retain for storage when not in use.
- Without stretching, place in direct contact with the 'at risk' area.
  - Either side of Fresco Pads may be applied to the skin.
  - Note that the heel, sacrum and tube are pre-shaped to allow for easier application.
- Use an appropriate secondary fixation such as tape, socks, bandages, tubular bandages or underwear to ensure Fresco Pads remains in place.
  - Fresco Pads are not affected by being fully covered.
- Do not use under compression garments i.e. multi-layer bandaging and compression hosiery.

#### CLEANING

Fresco Pads can be cleaned and reused and with regular maintenance, is reliable and long-lasting. These instructions have been written to help the user maximise the longevity of Pads but in no way, guarantee its lifetime. It is the user's responsibility to regularly inspect and check to ensure that it is performing optimally.

If visible signs of dirt and contamination are present, clean immediately using detergent and water. In the absence of visible contamination, clean using detergent and water weekly.  
Do not use abrasive cloths.

Rinse with clean water and wipe down with single-use, nonabrasive cloth and thoroughly dry.

#### END OF LIFE DISPOSAL

Cared for correctly, Fresco Pads is long-lasting and durable. At the end of its life, clean according to the instructions and dispose of with non-hazardous waste.  
Replace when visibly worn, becomes difficult to clean, or fails to provide complete coverage of the area at risk of pressure damage due to loss of integrity.

#### PACKAGING AND STORAGE

Fresco Pads are supplied in re-usable clamshells and cardboard cartons.  
Fresco Pads are supplied pre-packaged and should be intact upon receipt. Before use, carefully check for completeness and signs of damage. Damaged packages or products should not be used. Keep container dry. Keep in a cool, well-ventilated place. Exposure to direct sunlight or elevated temperatures over prolonged periods may lead to thermal degradation. Avoid storage under pressure or elevated temperatures to minimise degradation of the product.

#### HANDLING

Improper use or handling may lead to damage and possible malfunction. Check Fresco Pads before use to ensure it is in working order. Do not use damaged products. All Fresco Pads are latex-free.

#### MANUFACTURERS GUARANTEE

Fresco International 2005, S.A. as manufacturer warrants Fresco Pads against manufacturers defects up to the expiry date printed on the packaging.

#### COMPLIANCE

Fresco Pads conforms to MDD 93/42/EEC and MDR 2017/745 and the following standards: BS EN ISO 9001:2015 Quality Management Systems. BS EN 13485:2016 Medical Devices. Quality Management Systems. Requirements for Regulatory Purposes. BS EN 14971:2012 Medical Devices. Application of Risk Management to Medical Devices. BS EN ISO 15223-1:2016 Medical Devices. Symbols to be Used with Medical Device Labels, Labelling and Information to be Supplied. General Requirements. BS EN ISO 10993-5:2009 Biological Evaluation of Medical Devices. Tests for In Vitro Cytotoxicity. BS EN 10993-10:2013 Biological Evaluation of Medical Devices. Tests for Irritation and Skin Sensitization.

#### COMPLAINTS

Any user who has any complaints or who have experienced any dissatisfaction in the product quality, identity, durability, reliability, safety, effectiveness, and performance should notify the distributor.

If any Fresco Pads product ever malfunctions and may have caused or contributed to the death or severe injury of a patient, users should notify Fresco International 2005, S.A., his distributor and your local competent authority immediately by telephone, fax or written correspondence. When filing a complaint, provide the component's name and number, lot number(s), your name and address, the nature of the complaint and notification of whether or not a written report from the distributor is requested.

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Voor meer informatie kunt u contact opnemen met Fresco International 2005, S.A. op tel. +34 932 314 812